



**DR. WEIGERT**  
Systematic Hygiene

## neoform® wipes RTF: Instructions for filling with Dr. Weigert surface disinfectants.



1. Before preparing the ready-to-use disposable wipe dispensing system disinfect hands (e.g. with **weigoman® pure**) and wear protective gloves.



2. Removal of the wipe dispenser lid by opening the tamper-evident closure. Then slowly fill at least 2.25 L of the Dr. Weigert ready-to-use, alcoholic surface disinfectant or at least 2.5 L of the prepared use-solution \_\_\_\_\_ in accordance with the application recommendation.




3. Pull the first fleece wipe from the middle of the wipe roll through the opening of the dispensing lid, pull to the tapered end of the opening, tear off and discard.



4. Place the lid on the bucket and tightly close the opening. Fill in the preparation information on the wipe dispenser system (product name, batch, date of preparation, concentration, use-by date). After 30 min the fleece wipes are completely soaked and ready to use.

5. Wipe the surfaces to be disinfected carefully with the soaked **neoform® wipes**. For safe disinfection make sure the surfaces are completely wetted and observe the product-specific contact time. Then discard the fleece wipe.

6. Keep the wipe dispenser system tightly closed after taking out the **neoform® wipes**.

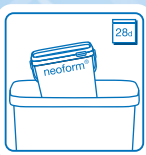

7. Disposal system: After depletion or after expiry of the shelf life, properly dispose of product residues and dispenser system. The soaked **neoform® wipes** can be used for up to 28 days if the dispenser system is properly closed.

**neoform® Rapid:** For medical areas, nursing homes and laboratories



**neoform® Classic:** For medical areas, nursing homes and laboratories



**The best solution  
for all areas:**



**neoform® K sprint:** For professional kitchens and the food industry



**neoform® K plus:** For professional kitchens and the food industry



**DR. WEIGERT**