



DR. WEIGERT
Hygiene mit System

neodisher endo® MED in combination with neoform® wipes RTF - for wipe cleaning during the bedside cleaning of flexible endoscopes



1. Before preparing the ready-to-use disposable wipe dispensing system disinfect hands (e.g. with **weigoman® pure**) and wear protective gloves.



2. Removal of the wipe dispenser lid by opening the tamper-evident closure. Then slowly fill in 2.5 L of a 1 % **neodisher endo® MED** application solution.



3. Pull the first fleece wipe from the middle of the wipe roll through the opening of the dispensing lid, pull to the tapered end of the opening, tear off and discard.



4. Place the lid on the bucket and tightly close the opening. Fill in the preparation information on the wipe dispenser system (product name, batch, date of preparation, concentration, use-by date). After 30 min the fleece wipes are completely soaked and ready to use.

5. For correct use, wipe the insertion tube of the endoscope to be cleaned carefully in the direction of the distal end with the soaked **neoform® wipes**. Then flush the channels of the endoscope with detergent solution, e.g. with a **neodisher endo® MED** application solution, and submit the endoscope for further processing. Discard the used fleece wipe.

6. Keep the wipe dispenser system tightly closed after taking out the **neoform® wipes**.



7. Disposal system: After depletion or after expiry of the shelf life, properly dispose of product residues and dispenser system. The soaked **neoform® wipes** can be used for up to 28 days if the dispenser system is properly closed.

neoform® wipes RTF: The efficient
wipe dispensing system



neodisher endo® MED: For the
manual pre-treatment of flexible
endoscopes