



DR. WEIGERT
Systematic Hygiene

Endoscopy



neodisher
endo® DIS
active

Manual Disinfecting Cleaning and
Manual Disinfection of Flexible Endoscopes.

neodisher endo® DIS active – the Granulate for Safe Manual Endoscope Reprocessing.

neodisher endo® DIS active

The granulate combines high-level disinfection with active cleaning.

- For the disinfecting cleaning of flexible endoscopes and endoscopic accessories in immersion baths and ultrasonic baths
- For the manual disinfection of flexible endoscopes and endoscopic accessories



8 kg Bucket
4077 76



4 x 2 kg Bucket
4077 81



25 x 100 g Sachet
4077 79

Easy to Use:

- Prepare desired water volume (max. lukewarm)
- Add granulate according to dosing table, stir until completely dissolved
- After 15 min the working solution is ready to be used
- To determine the concentration **neodisher® test strips** are available
- 1.0 % working solution (10 g/l) contains 0.15 % peracetic acid

Application Recommendation (20 °C):

Disinfecting Cleaning

bactericidal, yeasticidal, active against enveloped viruses (incl. e.g. HBV, HIV, HCV)

10 g/l (1.0 %),
5 min

Disinfecting Cleaning with Sporicidal Activity

bactericidal, yeasticidal, active against enveloped viruses (incl. e.g. HBV, HIV, HCV), sporicidal (incl. Clostridium difficile)

10 g/l (1.0 %),
15 min

Disinfection

bactericidal, mycobactericidal, fungicidal, virucidal (incl. e.g. rota virus, norovirus, HAV), sporicidal (incl. Clostridium difficile)

20 g/l (2.0 %),
15 min

50 strips

50 x neodisher® test strips
9813 20



Excellent material compatibility – suitable for endoscopes of all leading manufacturers

Our Competence – Your Benefit:

- extensive microbiological activity
- optimum cleaning performance
- pH neutral, free of perfumes, low foam formulation
- available in convenient 100 g sachets
- coated granulate
- free of aldehydes, amines and quaternary ammonium compounds



- VAH¹-listed, ÖGHMP²-listed, on the list of the IHO³ for disinfectants with virucidal activity
- no protein fixation
- particularly user-friendly, pleasant odour
- clean, precise and simple dosing
- low dust, easily soluble
- based on peracetic acid, no incompatibilities with other active substances

¹ Verbund für Angewandte Hygiene e.V. (Association of Applied Hygiene)

² Österreichische Gesellschaft für Hygiene, Mikrobiologie und Präventivmedizin [Austrian Society of Hygiene, Microbiology and Preventive Medicine]

³ Industrieverband Hygiene und Oberflächenschutz [Industry Association Surface Protection and Hygiene, Germany]