



DR. WEIGERT
Systematic Hygiene

Endoscopy



neodisher
endo[®] CLEAN
neodisher
endo[®] SEPT PAC

Innovative Automated Cleaning and Disinfection
of Flexible Endoscopes in Washer-Disinfectors.

neodisher endo[®] CLEAN – Guaranteed Excellent Cleaning Performance and Material Compatibility.

neodisher endo[®] CLEAN

The alkaline enzymatic cleaner combines outstanding cleaning performance and impeccable material compatibility.

- For the automated cleaning of flexible endoscopes and endoscopic accessories
- For the manual cleaning of flexible endoscopes and endoscopic accessories in immersion or ultrasonic baths



220 kg drum
4074 13



20 L canister
4074 26



10 L canister
4074 30



5 L canister
4074 33



6 x 2 L bottle
4074 42

Maximum Cleaning Performance:

- **neodisher endo[®] CLEAN** solves organic residues and prevents their redeposition. The detergent reliably removes residues of dried and denatured blood.
- **neodisher endo[®] CLEAN** fulfils the current recommendations of the Robert Koch Institute (RKI) for the reprocessing of medical devices to minimise the risk of transmitting the new variant of the Creutzfeldt-Jakob Disease (vCJD).

Application Recommendation:

automated cleaning of flexible endoscopes	5 ml/l (0.5 %), 35 - 55 °C, 3 - 5 min
automated cleaning of thermo-stable endoscopic accessories	2 - 10 ml/l (0.2 - 1 %) ¹ , 50 - 60 °C, 10 min
manual cleaning	5 - 10 ml/l (0.5 - 1 %) ¹ , max. 40 °C, 5 - 10 min

The cleaning performance must be verified in a validation on site.



Excellent material compatibility – suitable for endoscopes of all leading manufacturers

Our Competence – Your Benefit:

- unique formula based on alkali donors, enzymes and surfactants
- low-foam formulation
- excellent material compatibility
- fulfils the current reprocessing recommendations of the RKI to minimise the risk of transmitting vCJD
- usable for manual and automated cleaning



- excellent cleaning performance
- cleaning performance is not inhibited by foam development
- suitable for endoscopes of all leading manufacturers
- safe reprocessing of flexible endoscopes and endoscopic accessories
- one product for two reprocessing steps; when used for manual pre-cleaning the cleaner solution does not need to be rinsed off prior to the automated reprocessing step with neodisher endo[®] CLEAN

neodisher endo® SEPT PAC – Turbo Power for Modern Automated High-level Disinfection.

neodisher endo® SEPT PAC

The disinfectant based on peracetic acid stands out due to its rapid efficacy and extensive activity spectrum.

- For the automated disinfection of flexible endoscopes in washer-disinfectors (WD-E).



4.75 L canister
4075 35

Effective Disinfection:

- **neodisher endo® SEPT PAC** is active against bacteria (incl. MRSA, tuberculosis pathogens, Helicobacter pylori), fungi, mycobacteria, spores (incl. C. difficile) and viruses (incl. hepatitis A, B and C, HIV, rotaviruses, noroviruses). The disinfecting activity has been tested and confirmed according to the DIN EN 14885.
- **neodisher endo® SEPT PAC** complies with the requirements on disinfectants for the disinfection of flexible endoscopes according to the DIN EN ISO 15883-4.
- The automated reprocessing process with **neodisher endo® CLEAN** and **neodisher endo® SEPT PAC** fulfils the requirements of the DIN EN ISO 15883-4 regarding a germ reduction of > 9 log in the entire process.
- Active substances in 100 g: 15.0 g peracetic acid.

Application Recommendation:

automated disinfection of flexible endoscopes

10 ml/l (1.0 %),
25 °C, 10 min
or
10 ml/l (1.0 %),
35 °C, 5 min



weigomatic® endoDOS

The dosing system for an optimum and safe dosage when reprocessing endoscopes - for further information contact us!

Our Competence – Your Benefit:

- extensive activity at low temperatures
- free of aldehydes, amines and QAC, based on peracetic acid
- without CMR⁴ disinfecting active substances
- excellent material compatibility
- the products neodisher endo® CLEAN and neodisher endo® SEPT PAC are perfectly matched



- included in the IHO² list of disinfectants; fully active in only 5 min, incl. sporicidal activity (without activator), saves energy and resources
- no incompatibilities with other active substances, environmentally friendly degradation products of the active substance peracetic acid
- very user-friendly
- suitable for endoscopes of all leading manufacturers
- a possible carry-over of neodisher endo® CLEAN into the disinfection step has no impact on the disinfection performance of neodisher endo® SEPT PAC

For further information on the products and application please refer to the respective product data sheet.

¹The dosing amount depends on the degree of soiling.

²Industrieverband Hygiene und Oberflächenschutz [Industry Association Surface Protection and Hygiene, Germany] ³Quaternary Ammonium Compounds

⁴Carcinogenic, Mutagenic, toxic to Reproduction



DR. WEIGERT
Systematic Hygiene



Always Nearby to Answer Your Hygiene Questions – in Europe and Around the World:

Chemische Fabrik
Dr. Weigert GmbH & Co. KG
Mühlenhagen 85
20539 Hamburg
Tel.: +49-40-7 89 60-0
info@drweigert.de

Dr. Weigert
Handelsgesellschaft m.b.H.
Wienerbergstraße 11/12 a
1100 Wien
Tel.: +43-1-99460 6275
info@drweigert.at

Dr. Weigert Nederland BV
Balkendwarsweg 3
9405 PT Assen
Tel.: +31-592-31 93 93
info@drweigert.nl

Dr. Weigert Hungária Kft.
Hunyadi János út 16
1117 Budapest
Tel.: +36-1-237 06 04
Info@drweigert.hu

Dr. Weigert Polska Sp. z o.o.
ul. Wyrżeże Gdyńskie 6D
01-531 Warszawa
Tel.: +48-22-616 02 23/31
office_pl@drweigert.com

Dr. Weigert (Schweiz) AG
Baarerstrasse 14
6300 Zug
Tel.: +41-41 561 32 90
info@drweigert.ch

Dr. Weigert Belgium N.V.
Bisschop Ruttenstraat 17
3640 Kinrooi-Geistingen
Tel.: +32-89 56 49 25
info@drweigert.be

Dr. Weigert UK Ltd
Newmarket Suite 17
Keys Business Village
Keys Park Road
Cannock, WS12 2HA
Tel.: +44-1543 471 222
enquiry@drweigert.com

Dr. Weigert France SAS
22 avenue des Nations
Immeuble le Raspail
BP 88035 Villepinte
95932 Roissy CDG Cedex
Tel.: +33-1-48 67 90 33
info@drweigert.fr

Dr. Weigert España
Edificio MB-One
Avda. de Europa, 19/3A
28108 Alcobendas Madrid
Tel.: +34-91 793 21 48
info.spain@drweigert.com

Dr. Weigert Management System – Certified according to DIN EN ISO 9001/13485/14001

www.drweigert.com

