



DR. WEIGERT
Systematic Hygiene

Medical | Laboratory | Pharma & Cosmetic



neoform[®]
Rapid¹

Alcohol-based rapid disinfectant for surfaces in the medical, laboratory, pharmaceutical and cosmetics sectors


neiform® Rapid¹ –

Rapid disinfectant for surfaces in the medical, laboratory, pharmaceutical and cosmetics sectors

neiform® Rapid¹

Alcohol-based ready-to-use solution for targeted and rapid disinfection of

- Surfaces of medical devices such as surgical tables and other surfaces in hospitals and doctors' and dentists' practices
- Surfaces and equipment in the pharmaceutical and cosmetics sectors
- Surfaces in laboratories, e.g. laboratory benches
- Bactericidal, yeasticidal, fungicidal, tuberculocidal, active against enveloped viruses and limited spectrum virucidal activity (effective against adenoviruses and noroviruses), effective against rotaviruses
- The surfaces to be disinfected must be compatible with alcohol-based disinfectants

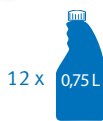

12 x 0,75 L

neiform® Rapid:
12 x 0,75 L bottle
5643 47

Application:

- Apply **neiform® Rapid** undiluted to clean, dry surfaces, ensuring that the surfaces are wetted, and wipe the surfaces to be disinfected
- Adhere to the contact times below for the desired disinfecting activity
- Also suitable for use with the Dr. Weigert fleece wipe dispensing system **neiform** wipes RTF
- Dries rapidly without leaving residues, no rinsing required
- Broad activity spectrum, even after short contact times
- Active substances in 100 g:
35,0 g propan-1-ol und 25,0 g ethanol

Application recommendation (20 °C):

Disinfecting activity
(clean conditions, pre-cleaned surfaces)

Bactericidal (VAH ² , EN 13727, EN 16615)	Undiluted, 1 min
Yeasticidal (VAH ² , EN 13624, EN 16615)	Undiluted, 1 min
Fungicidal (EN 13697)	Undiluted, 3 min
Tuberculocidal (EN 14348)	Undiluted, 30 sec
Active against enveloped viruses (incl. HBV, HCV, HIV) (EN 14476) (RKI/DVV) ³	Undiluted, 15 sec
Limited spectrum virucidal activity (adenovirus, norovirus, incl. all enveloped viruses) (EN 14476)	Undiluted, 2 min
Active against rotaviruses (EN 14476)	Undiluted, 1 min
VAH ² -list	Undiluted, 5 min

neiform® Rapid acts quickly and dries without leaving residues. Active against noroviruses in only 2 minutes!

Our Competence – Your Benefit:

- Short contact time, broad activity spectrum
- Free from aldehydes, colouring agents and perfumes
- Very user-friendly
- Limited spectrum virucidal activity



- VAH²-listed, included in the IHO⁴ disinfectant list
- No rinsing required
- The application in the Dr. Weigert fleece wipe dispensing system **neiform** wipes RTF fulfils the requirements of the current VAH recommendation⁵
- Can be used for rapid disinfection during Norovirus outbreaks

¹ Use disinfectant with safety. Always read the label and product information before use.

² Association of Applied Hygiene

³ According to the test guidelines of the Robert Koch Institute (RKI) and the German Association for the Control of Virus Diseases (DVV)

⁴ Industrial Association for Surface Protection and Hygiene, Germany

⁵ Recommendation for checking critical points when using wipes dispenser systems in a pre-soak system for surface disinfection (Hyg Med 2012; 37:11)