

neodisher[®] FRN

Liquid, pH-neutral cleaning agent
for use in decontamination units, free from phosphates



Main fields of application: Mechanical cleaning of bedframes, trolleys and containers in decontamination units.

Characteristics: neodisher FRN pH-neutral is a detergent for sensitive materials based on special surfactants in non-foaming formulation. neodisher FRN is free from oxidising agents, acids and alkalis, so that the neutrality of the effluent is guaranteed.

Sensitive materials on bedframes, containers, etc., are not attacked.

Can be used in soft water up to 5 °d (1 mmol/l) total hardness.

Application and dosage: 1 - 5 ml / l using suitable dosing devices

Subsequently the neodisher FRN-solution has to be rinsed out completely by sufficient rinsing. Do not mix with other products.

Technical data:

Density (20 °C): 1.0 g/cm³

pH values of neodisher FRN-solutions, determined in water of 15 °d (3 mmol/l) softened to 0 °d (0 mmol/l), at 20 °C:

<u>Concentration neodisher FRN</u>	<u>pH value</u>
0 ml/l	7.5
1 ml/l	7.5
5 ml/l	7.5

Viscosity (concentrate, 20 °C): < 50 mPas

Ingredients:

Ingredients according to Regulation (EC) No 648/2004 on detergents:

15 - 30 % non-ionic surfactants

also preservation agents (N-(3-aminopropyl)-N-dodecylpropane-1,3-diamine)

CE-marks:

neodisher FRN conforms to the requirements of the legislation for medical devices according to the European directive 93/42/EC.

Storage information:

Bei der Lagerung ist eine Temperatur zwischen 0 und 30 °C einzuhalten. Usable for 4.years when stored as recommended.

Hazard and precautionary statements:

neodisher FRN is not a hazardous product according to CLP Regulation (EC) No. 1272/2008. Dispose only when container is empty and closed. For disposal of product residues, refer to the Material Safety Data Sheet.

For further safety information see EC safety data sheets. These are available at www.drweigert.com under the category "Download".

MB 4120 GB/2-2
10/14



DR. WEIGERT

Chemische Fabrik Dr. Weigert GmbH & Co. KG
Mühlenhagen 85, D – 20539 Hamburg

Telefon: (040) 789 60 - 0
Telefax: (040) 789 60 - 120

E-Mail: info@drweigert.de
Internet: www.drweigert.de

Die Angaben dieses Merkblattes beruhen auf unseren derzeitigen Kenntnissen und Erfahrungen. Sie befreien den Verwender nicht von eigenen Prüfungen und Versuchen. Eine rechtlich verbindliche Zusicherung bestimmter Eigenschaften kann hieraus nicht abgeleitet werden.

With the above information, which is appropriate to our current knowledge we describe our product regarding possible safety necessities, but we do not involve any quality description or promise certain properties.