

neodisher[®] IP Konz

Liquid lubricant for use in special washing machines.
Toxicologically certified



Main fields of application:

Mechanical lubrication of surgical instruments in the final rinse in special washing machines.

Characteristics:

neodisher IP Konz is a lubricant based on highly purified liquid paraffin and emulsifiers in accordance with the official German pharmacopoeias. neodisher IP Konz is added to the final rinse water of washing machines for instrument lubrication and disperses as a fine emulsion. Remaining residues on instrument surfaces do not resinify during the sterilisation process. Because of its emulsifier content, there is no interference with steam sterilisation. In subsequent cleaning the film is removed again without residue. neodisher IP Konz is not suitable for instruments sterilized with hot air. neodisher IP Konz is subject to a final bacteriological check and is free from vegetative germination.

Application and dosage:

In special washing machines for surgical instruments:

1 - 2 ml/l using suitable dosing devices, added to the final machine rinse water of the washing programme. The water for the final rinse should have a temperature of at least 50 °C.

We do not recommend neodisher IP Konz for use in immersion baths because of the danger of uncontrolled introduction of germs.

Alternatively, neodisher IP Spray may be used for treatment after automated reprocessing.

Do not mix with other products.

Reprocessing should comply with all ordinances pursuant to the Medical Device Directive and should be performed with appropriate validated processes.

Expert report/listings:

If applied as intended the product is toxicologically safe (assessment in accordance with EN ISO 10993-1). A summary of the toxicological profile is available on request. The active agents are FDA-registered (No.176210, 177120 and 172878).

Technical data:

Density (20 °C): 0.9 g/cm³

Viscosity (concentrate, 20 °C): < 50 mPas

Ingredients:

Ingredients according to Regulation (EC) No 648/2004 on detergents:

15 - 30 % non-ionic surfactants

> 30 % aliphatic hydrocarbons (liquid paraffin DAB)

also preservatives

CE-marks:



neodisher IP Konz conforms to the European directive 93/42/EEC, Annex I, concerning medical devices.

Storage information:

Protect from frost. At temperatures below 10 °C reversible clouding can occur. Usable for 2 years when stored as recommended.

Hazard and precautionary statements:

For safety information see EC Safety Data Sheets. These are available at www.drweigert.com under the category "Service".

Dispose only when container is empty and closed. For disposal of product residues, refer to Safety Data Sheet.

MB 4306 GB/2-4 05/15



Chemische Fabrik Dr. Weigert GmbH & Co. KG
Mühlenhagen 85, D - 20539 Hamburg

Telefon: (040) 789 60 - 0
Telefax: (040) 789 60 - 120

E-Mail: info@drweigert.de
Internet: www.drweigert.de

Die Angaben dieses Merkblattes beruhen auf unseren derzeitigen Kenntnissen und Erfahrungen. Sie befreien den Verwender nicht von eigenen Prüfungen und Versuchen. Eine rechtlich verbindliche Zusicherung bestimmter Eigenschaften kann hieraus nicht abgeleitet werden.

With the above information, which is appropriate to our current knowledge we describe our product regarding possible safety necessities, but we do not involve any quality description or promise certain properties.