




neodisher[®] SB eco

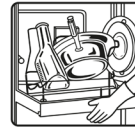

Special rinse aid for use in washer-disinfectors for bedpans



Liquid concentrate

Fields of application:

- Complexing water hardness and rinsing when reprocessing bedpans and urine bottles in bedpan washer-disinfectors with steam disinfection

Performance spectrum:

- Reliably prevents lime residues in steam generators, pipes and jets
- Mildly acidic formula for effectively complexing water hardness

Special properties:

- For stain-free bedpans and urine bottles without limescale deposits
- Suitable for all degrees of water hardness
- Mildly acidic, based on organic acids

Application and dosage:

neodisher SB eco is used in bedpan washer-disinfectors with steam disinfection. neodisher SB eco is dosed using dosing equipment installed in the washer-disinfector. The dosing amount is 4 - 12 ml per cycle depending on water hardness with a water volume of 8 l. Dosing systems and steam generators must be acid-compatible.

For removing stubborn soiling we recommend the additional use of neodisher SBR extra, a special detergent for bedpans and urine bottles.

Notes on application:

- For professional use only.
- Rinse out dosing system including suction hose with water before changing product.
- Reprocessing should comply with all ordinances pursuant to the regulations on medical devices and should be performed with appropriate validated processes.
- Please observe the reprocessing recommendations of the medical device

manufacturers according to the requirements of the DIN EN ISO 17664.

- The instructions given by the manufacturer of the washer disinfector are to be observed.
- Do not mix with other products.

Technical data:

pH-range	3.7 - 3.0 (0.5 - 1.5 ml/l, determined in deionised water, 20 °C)
Viscosity	< 50 mPa s (concentrate, 20 °C)
Density	approx. 1.1 g/cm ³ (20 °C)

Ingredients:


Ingredients according to Regulation (EC) No 648/2004 on detergents:
< 5 % polycarboxylates
also preservatives (potassium sorbate, chloromethylisothiazolone/methylisothiazolone)

CE-marks:

neodisher SB eco complies with European guidelines for medical devices.

If a serious incident occurs with the product, report it to the manufacturer and the relevant national authority.

Storage information:

Always store at a temperature between 0 °C and 30 °C. Usable for 3 years when stored as recommended. For expiry date refer to the stamp mark on the label behind the hourglass symbol .

Hazard and precautionary statements:

For safety information see Safety Data Sheets. These are available e.g. at www.drweigert.com under the headline "Service/Downloads".

Dispose only when container is empty and closed. For disposal of product residues, refer to the Safety Data Sheet.

MB 4067/3-3 Date of issue: 03/2020