




neodisher[®] SBR extra



Detergent for use in washer disinfectors for bedpans

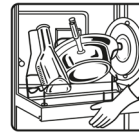

Liquid concentrate

Fields of application:

- Automated cleaning of bedpans and urine bottles in bedpan washer disinfectors

Performance spectrum:

- Removes even stubborn stains such as saponified lime residues and residues from cytostatic agents from human excrements

Special properties:

- Low-foam formulation
- Can be used in water of any hardness
- Also usable in suitable bedpan washer disinfectors for the complexing of water hardness and rinsing

Application and dosage:

neodisher SBR extra can be used in bedpan washer disinfectors that are equipped with a dosing pump for detergents. The dosage is 1 - 3 ml/l depending on the water hardness and the degree of contamination in the cleaning step.

Notes on application:

- For professional use only.
- Rinse out dosing system including suction hose with water before changing product.
- The neodisher SBR extra-solution has to be rinsed off completely with water.
- Reprocessing should comply with all ordinances pursuant to the regulations on medical devices and should be performed with appropriate validated processes.
- Please observe the reprocessing recommendations of the medical device manufacturers according to the requirements of the DIN EN ISO 17664.
- The instructions given by the manufacturer of the washer disinfectant are to be observed.
- Do not mix with other products.

Technical data:

pH range	11.2 - 11.5 (1 - 3 ml/l, determined in deionised water, 20 °C)
Viscosity	< 50 mPa s (concentrate, 20 °C)
Density	approx. 1.2 g/cm ³ (20 °C)

Ingredients:


Ingredients according to Regulation (EC) No 648/2004 on detergents:
< 5 % phosphonates
15 - 30 % EDTA

CE-mark:

neodisher SBR extra complies with European guidelines for medical devices.

If a serious incident occurs with the product, report it to the manufacturer and the relevant national authority.

Storage information:

Always store at a temperature between 0 °C and 30 °C. Usable for 3 years when stored as recommended. For expiry date refer to the stamp mark on the label behind the hourglass symbol .

Hazard and precautionary statements:

For safety information see Safety Data Sheets. These are available at www.drweigert.com under the category "Service/Downloads".

Dispose only when container is empty and closed. For disposal of product residues, refer to the Safety Data Sheet.

MB 4007/3-2
Date of issue: 10/2021