




neodisher[®] Septo DN 2



Disinfectant for the automated reprocessing of flexible endoscopes

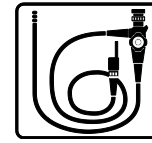

Liquid concentrate

Fields of application:

- For the disinfection of flexible endoscopes in washer disinfectors

Performance spectrum:

- Bactericidal, fungicidal, mycobactericidal and virucidal activity according to EN standards. Confirmed by certification
- On the IHO¹ list of disinfectants with virucidal activity

Special properties:

- Wide activity spectrum
- Excellent material compatibility
- Free of formaldehyde and quaternary ammonium compounds

Application and dosage:

neodisher Septo DN 2 is dosed using the machine's integrated dosing device at the beginning of the disinfection step.

bactericidal, fungicidal, mycobactericidal and virucidal activity	1.0 %, 55 °C, 5 min.
---	----------------------

In the cleaning step the detergent neodisher MediClean forte is recommended.

General instructions on use:

- For professional use only.
- The neodisher Septo DN 2 working solution has to be rinsed off completely with water (preferably deionised water).
 - For avoiding water stains the use of deionised water in the final rinse is recommended.
- Rinse out dosing system including suction hose with water before changing product.
- Please observe the instrument manufacturer's recommendations for reprocessing according to the requirements of DIN EN ISO 17664.
- The instructions given by manufacturer of the washer disinfectors are to be observed.
- Do not mix with other products.



neodisher[®] Septo DN 2

Expert reports:

The disinfecting activity has been confirmed by certification.
Expert reports are available on request.

Technical data:

pH-value	approx. 7 (1.0 %, determined in deionised water (from 15°d [2.7 mmol/l] to 0°d [0 mmol/l]), 20 °C)
Viscosity	< 10 mPa s (concentrate, 20 °C)
Density	1.05 g/cm ³ (20 °C)

Hazard warnings and safety advice:

For safety information see EC safety data sheets. These are available e.g. at www.drweigert.de under the category "Service".

Dispose only when container is empty and closed. For disposal of product residues, refer to Material Safety Data Sheet.

MB 4990/3-2
Date of Issue: 04/2013


Ingredients:

Ingredients according to Regulation (EC) No 648/2004 on detergents:
< 5 % phosphonates
disinfecting agent in 100 g: 20.0 g glutaral

CE-mark: 0297

neodisher Septo DN 2 conforms to the European directive 93/42/EC, Annex I, concerning medical devices.

Storage information:

Always store at a temperature between 0 and 30 °C.
Usable for 3 years when stored as recommended.
For expiry date refer to the stamp mark on the label behind the hour glass symbol .

With the above information, to our current knowledge we describe our product regarding safety necessities, but we do not involve any quality description or promise certain properties.